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TOSOH BIOSCIENCE LIMITED  
Lytchett House, 13 Freeland Park,  
Wareham Road, Poole, Dorset, BH16 6FA UK  
Phone : +44 1527 592901  
Fax : +44 1527 471680

TOSOH BIOSCIENCE, INC.  
3600 Gantz Road,  
Grove City, OH 43123 USA  
Phone : +1 650 615 4970  
Fax : +1 650 615 0415

<b>IVD</b> In vitro diagnostic medical device	<b>i</b> Consult instructions for use	<b>LOT</b> Batch code / Lot number
<b>Temperature limitation</b>	<b>Use-by date</b>	<b>REF</b> Catalogue number / Part number
<b>Manufacturer</b>	<b>EC REP</b> Authorised representative in the European Community	<b>NET</b> Net volume (after reconstitution for lyophilized material)
<b>UK RP</b> UK Responsible person	<b>UK CA</b> United Kingdom Conformity Assessed	<b>Supplied by</b> Supplied by
<b>UDI</b> A carrier containing unique device identifier information		

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### Attention

For U.S., Latin America, and Canada, please refer to the Tosoh Bioscience website at [www.tosohbioscience.us](http://www.tosohbioscience.us) for your country specific AAM-IFU. This Instructions for Use is not for use in the Americas.

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### INTENDED USE

The AIA-PACK ACTH CONTROL SET is intended for *IN VITRO* DIAGNOSTIC USE ONLY, by healthcare professionals, for performing quality control procedures with the ST AIA-PACK ACTH assay and the CL AIA-PACK ACTH assay.

### SUMMARY AND EXPLANATION

The AIA-PACK ACTH CONTROL SET contains buffered bovine serum albumin with the assigned levels of adrenocorticotrophic hormone (ACTH). The ACTH controls shall be run at the beginning of each day on which ACTH assays are scheduled.

### CONTENTS

- Cat. No. 0025421
- 2 x 1 mL AIA-PACK ACTH CONTROL LEVEL 1  
Buffered bovine serum albumin containing ACTH (Lyophilized). See the Data Sheet for the assigned concentration range.
- 2 x 1 mL AIA-PACK ACTH CONTROL LEVEL 2  
Buffered bovine serum albumin containing ACTH (Lyophilized). See the Data Sheet for the assigned concentration range.

### WARNINGS AND PRECAUTIONS

- The AIA-PACK ACTH CONTROL SET is intended for *in vitro* diagnostic use only.
- Inspect the packaging and the exterior of the vials for any sign of damage before use. If any damages are visible, contact your local Tosoh sales representative.
- Although material derived from human origin is not used for these controls, it is recommended that this product be handled with the same precautions as used for patient samples.
- Do not use beyond the expiration date.
- For safe waste disposal, it is recommended that each laboratory complies with established laboratory procedures and local, state, and federal regulations.

### PREPARATION OF REAGENTS

Bring the controls to 18-25 °C prior to use. Using volumetric pipettes, reconstitute the lyophilized controls accurately to the volume of 1 mL with CAP Class I water or the clinical laboratory reagent water (formerly NCCLS Type I) defined by CLSI GP40-A4-AMD guideline. Allow the lyophilized material to fully dissolve.

### STORAGE AND STABILITY

Always store the AIA-PACK ACTH CONTROL SET in an upright position at 2-8 °C when not in use. When stored unopened and refrigerated at 2-8 °C, the control set is stable until the expiration date on the label. The controls should be used within 7 days of opening or reconstituting, provided the vials are kept tightly sealed and refrigerated at 2-8 °C.

### PROCEDURE

Refer to the Tosoh AIA® System Operator's Manual or the Tosoh AIA-CL Analyzer Operator's Manual for additional procedural instructions regarding quality control.

- Load the appropriate amount of test cups on the instrument.
- Add the appropriate amount of each control to sample cups. (Refer to the instrument worksheet for the sample volume.)
- Print a worklist and place the sample cups in the position indicated.
- Select START. Verify that the positions on the worklist match the starting position on the screen.

### ASSIGNMENT OF VALUES

The AIA-PACK ACTH CONTROL SET contains assigned concentration range of ACTH. The assigned range is determined on a lot-by-lot basis and is designed to provide target control levels of ACTH. Since the assay values are dependent upon assay procedures as well as several other factors, each laboratory should establish its own range for the assay procedure being monitored.

### LIMITATIONS

The AIA-PACK ACTH CONTROL SET is designed solely for use with ST AIA-PACK ACTH and CL AIA-PACK ACTH assay procedures.

"AIA" and "AIA-PACK" are registered trademarks of Tosoh Corporation in the European Union, etc.