

AIA-360 Assay Specifications

ST LH II Test Code 039

No.	Item	Data
1	Calib. Req	Show
2	Cal 1	0 mIU/mL
3	Cal 2	5 mIU/mL (example)
4	Cal 3	20 mIU/mL (example)
5	Cal 4	50 mIU/mL (example)
6	Cal 5	100 mIU/mL (example)
7	Cal 6	200 mIU/mL (example)
8	Cal Lot L	
9	Cal Lot R	
10	Unit	mIU/mL
11	Smpl. Vol	40
12	Dil. Vol	100
13	Assay L	0.2
14	Assay H	200
15	Ref. L	
16	Ref. H	
17	Decimal	1

Visible only in Test Mode

18	Test Name	#LH2
19	Calib. No	6
20	Calib. Mul	3
21	Calib. Equ	4
22	Calib. CV	90
23	Assay Prtl	1
24	Factor1 A	1.000000
25	Factor1 B	0.000000
26	Factor2 A	1.000000
27	Factor2 B	0.000000
28	V. Conc	0.000000
29	G Origin	0.000000

AIA-900 Assay Specifications

ST LH II Test Code 039

No.	Item	Data
1	Code	39
2	ACT	1
3	Analyte	#LH2
4	Lot	***Enter current cal lot no.
5	CAL 1	0 mIU/mL
6	CAL 2	5 mIU/mL (example)
7	CAL 3	20 mIU/mL (example)
8	CAL 4	50 mIU/mL (example)
9	CAL 5	100 mIU/mL (example)
10	CAL 6	200 mIU/mL (example)
11	Cal lot L	
12	Cal lot R	
13	Unit	mIU/mL
14	Decimal	1
15	Assay Low	0.2
16	Assay High	200
17	Reference Low	
18	Reference High	
19	Reschedule Low	0.2
20	Reschedule High	200
21	Factor A	1
22	Factor B	0
23	Sample Volume	40
24	Diluent Volume	100
25	2 Step reagent dispensing volume	0
26	Calibration code	39
27	CAL. No.	6
28	CAL. MUL.	3
29	CAL. EQU.	4
30	CAL. CV	90
31	DIL. SP1	1
32	DIL. SP2	1
33	DIL. CAL. (Calculation of dil ratio of conc.)	1
34	DIL. CNTL.	1
35	DIL. DO	10
36	DIL. AH.	5
37	DIL. CALC.	1
38	DIL. CODE	36
39	DIL. NAME	LH2
40	DIL. PRTY	3
41	PRE. SPVOL (Vol. of pretreated sample)	0
42	PRE. 1VOL (Vol. of pretreatment sol-1)	0
43	PRE. 2 VOL (Vol. of pretreatment sol-2)	0
44	PRE. CODE (pretreated sol. code)	0
45	PRE. NAME (pretreated sol. name)	0
46	Protocol	1
47	SYS. F_A	1
48	SYS. F_B	0
49	V. CONC.	0
50	G. ORIGIN	0

AIA-2000 Assay Specifications

ST LH II Test Code 039

No.	Item	Data
1	Unit	mIU/mL
2	Decimal places	1
3	Reference low	
4	Reference high	
5	Reschedule low	0.2
6	Reschedule high	200
7	Assay range low	0.2
8	Assay range high	200
9	Specimen diluent code	36
10	Specimen diluent name	LH2
11	Dilution factor for Sp. 1	1
12	Dilution factor for Sp. 2	1
13	Dilution factor for Control	1
14	Default multiplier for DO	100
15	Default multiplier for >H	10
16	Factor A	1.00000e+000
17	Factor B	0.00000e+000
18	Calibration code	4
19	Calibrator replicates	3
20	Calibrator lot	***Enter current cal. lot no.
21	Calibrator Concentration	
22	Cal - 01	0 mIU/mL
23	Cal - 02	5 mIU/mL (example)
24	Cal - 03	20 mIU/mL (example)
25	Cal - 04	50 mIU/mL (example)
26	Cal - 05	100 mIU/mL (example)
27	Cal - 06	200 mIU/mL (example)
28		
29		
30		
31		
32		
33		
34	Dilution factor for calibrator	1
35	Calibration period	90
36	Incubation time (10/40)	10
37	Assay protocol	1
38	Specimen volume	40
39	Diluent volume	100
40	Conjugate volume	0
41	Diluted conjugate volume	0
42	Pretreatment	0
43	Pretreatment specimen volume	0
44	Pretreatment Reagent code	0
45	Pretreatment Reagent name	0
46	Pretreatment Reagent 1 volume	0
47	Pretreatment Reagent 2 volume	0
48	System Factor	1
49	Calibration Code Check	039
50	Virtual Concentration	0.0
51	Graph Origin	0.0
52	Calculation with Dilution Factor	Yes

Luteinizing Hormone ST AIA-PACK LH II

Name and Intended Use

ST AIA-PACK LH II is designed for IN VITRO DIAGNOSTIC USE ONLY for the quantitative measurement of Luteinizing Hormone (LH) in human serum or heparinized plasma on specific Tosoh AIA System analyzers.

Summary and Explanation of Test

Human Luteinizing Hormone (LH) is a glycoprotein which, like FSH, HCG and TSH consists of alpha and beta chains. The Alpha chain is virtually identical in all four hormones, whereas the beta chains are different and determine both the specific biological activity and immunological characteristics of each hormone.^{1,2}

The LH molecule contains approximately 15-18% carbohydrate and has a molecular weight of approximately 29,000 daltons.^{3,4} LH induces ovulation and thereafter maintains the corpus luteum and progesterone production in women.⁵⁻⁸ In men, LH stimulates testosterone production by the Leydig cells.^{5,6,9}

Hypothalamic control of both LH and FSH secretion by the anterior pituitary appears to be by a common releasing hormone, gonadotropin-releasing hormone (GnRH) or luteinizing hormone-releasing hormone (LHRH), with negative feedback at the hypothalamic level by estrogen in the female and testosterone in the male.

Through the use of monoclonal antibody technology which provides the necessary specificity and sensitivity, determination of LH concentrations is critically important in assessment and monitoring of patients with suspected infertility.^{10,11} Raised levels of LH and LH:FSH ratio are frequently found in polycystic ovary syndrome (PCOS), a common cause of infertility in women and other physiological disorders associated with abnormal LH secretion.¹²⁻¹⁵

Principle of the Assay

The ST AIA-PACK LH II is a two-site immunoassay which is performed entirely in the AIA-PACK. LH present in the test sample is bound with monoclonal antibody immobilized on a magnetic solid phase and enzyme-labeled monoclonal antibody in the AIA-PACK. The magnetic beads are washed to remove unbound enzyme-labeled monoclonal antibody and are then incubated with a fluorogenic substrate, 4-methylumbelliferyl phosphate (4MUP). The amount of enzyme-labeled monoclonal antibody that binds to the beads is directly proportional to the LH concentration in the test sample. A standard curve is constructed, and unknown sample concentrations are calculated using this curve.

Material Provided (ST AIA-PACK LH II, Cat. No. 0025296)

5 trays x 20 test cups (ST AIA-PACK LH II Test Cup)

Plastic test cups containing lyophilized magnetic beads coated with anti-LH mouse monoclonal antibody and mouse monoclonal antibody (to human LH) conjugated to bovine alkaline phosphatase with 0.1% sodium azide as a preservative.

Materials Required But Not Provided

The following materials are not provided but are required to perform Luteinizing Hormone analysis using the ST AIA-PACK LH II (Cat. No. 0025296) on specific Tosoh AIA Systems. They are available separately from Tosoh.

Materials	Cat. No.
AIA-SYSTEMS:	
AIA-360	0019945
AIA-900	0022930
AIA-900 9tray Sorter	0022931
AIA-900 19tray Sorter	0022932
AIA-2000 ST	0022100
AIA-2000 LA	0022101
AIA-PACK:	
AIA-PACK Substrate Set II	0020968
AIA-PACK Substrate/Reconstituent	
ST AIA-PACK LH II Calibrator Set	0025396
Calibrator #1 0 mIU/mL	
Calibrator #2 5 mIU/mL (approx.)	
Calibrator #3 20 mIU/mL (approx.)	
Calibrator #4 50 mIU/mL (approx.)	
Calibrator #5 100 mIU/mL (approx.)	
Calibrator #6 200 mIU/mL (approx.)	
AIA-PACK LH II Sample Diluting Solution	0020596
AIA-PACK Wash Concentrate Set	0020955
AIA-PACK Diluent Concentrate Set	0020956
AIA-PACK Detector Standardization Test Cups	0020970
AIA-PACK Sample Treatment Cups	0020971
Sample Cups	0018581
ADDITIONAL REQUIREMENTS: (Except AIA-360)	
Pipette Tips (1000/Pkg)	019215
Tip Rack (Empty)	019216
Preloaded Pipette Tips (96 Tips X 50 Racks)	996010
Preloaded Pipette Tips (96 Tips X 5 Racks)	996005

Only materials obtained from Tosoh should be used. Materials obtained elsewhere should not be substituted since assay performance is based strictly on Tosoh materials.

Warnings and Precautions

- The ST AIA-PACK LH II is intended for in vitro diagnostic use only.
- **Rx** only. US Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.
- Inspect the packaging and the exterior of the aluminum pouch or the vials for any sign of damage before use. If any damages are visible, contact your local Tosoh sales representative.
- Test cups from different lots should not be mixed within a tray,
- The ST AIA-PACK LH II contains sodium azide, which may react with lead or copper plumbing to form potentially explosive metal azides. When disposing of such reagents, always flush with large volumes of water to prevent azide build-up.
- Human sera is not used in the preparation of this product, however, since human specimens will be used for samples and other quality control products in the lab may be derived from human serum, please use standard laboratory safety procedures in handling all specimens and controls.
- For safe waste disposal, it is recommended that each laboratory complies with established laboratory procedures and local, state, and federal regulations.
- Do not use beyond the expiration date.
- The ST AIA-PACK LH II has been designed so that the high dose “hook effect” is not a problem for the vast majority of samples. Samples with LH concentrations between 200 and 10,000 mIU/mL will read > 200 mIU/mL. The “hook effect” phenomenon may occur at LH concentrations > 10,000 mIU/mL.
- Serum, dust, metal, or microorganism contamination may cause degradation of reconstituted substrate solution. Store in a clean environment, away from direct sunlight and ultraviolet light.
- Tosoh Automated Immunoassays utilizing alkaline phosphatase-based technologies should not be used with samples from patients under Asfotase Alfa treatment.

Storage and Stability

All unopened materials are stable until the expiration date on the label when stored at the specified temperature.

Materials	Cat. No.
Refrigerator Temperature (2 - 8 °C):	
ST AIA-PACK LH II	0025296
ST AIA-PACK LH II Calibrator Set	0025396
AIA-PACK LH II Sample Diluting Solution	0020596
AIA-PACK Substrate Set II	0020968
AIA-PACK Wash Concentrate	0020955
AIA-PACK Diluent Concentrate	0020956

Room Temperature (18 - 25 °C):

AIA-PACK Detector Standardization Test Cups

0020970

AIA-PACK Sample Treatment Cups

0020971

AIA-PACK test cups may be stored for up to 24 hours at a room temperature of 18 - 25 °C. Calibrators are stable for up to 24 hours after opening, provided the vials are kept tightly sealed and refrigerated at 2 - 8 °C. After opening, Sample Diluting Solution is stable for up to 7 days if kept tightly sealed and refrigerated at 2 - 8 °C. Reconstituted substrate solution is stable for 3 days at 18 - 25 °C or 30 days at 2 - 8 °C. Working diluent and wash solutions are stable for 30 days at room temperature (18 - 25 °C). Reagents should not be used if they appear cloudy or discolored.

Specimen Collection and Handling

Serum or heparinized plasma is required for the assay. EDTA and citrated plasma **SHOULD NOT BE USED**.

No special patient preparation is necessary. When using serum, a venous blood sample is collected aseptically without additives. Store at 18 - 25 °C until a clot has formed (usually 15 - 45 minutes), then centrifuge to obtain the serum specimen for assay.

To use heparinized plasma, a venous blood sample is collected aseptically with the designated additive. Centrifuge and separate plasma from the packed cells as soon as possible.

Samples may be stored at 2° - 8 ° C for up to 24 hours prior to analysis. If the analysis cannot be done within 24 hours, the sample should be stored frozen at -20 °C or below for up to 60 days.

Repeated freeze-thaw cycles should be avoided. Turbid serum samples or samples containing particulate matter should be centrifuged prior to testing. Prior to assay, slowly bring frozen samples to room temperature (18 - 25 °C) and mix gently.

Diluted sample should be used within 24 hours.

The sample required for analysis is 40 µL.

Procedure

1) Reagent Preparation

1a) Substrate Solution

Bring all reagents to room temperature (18 - 25 °C) before preparing the working reagent. Add the entire contents of the Substrate Reconstituent (100 mL) to the lyophilized Substrate and mix thoroughly to dissolve the solid material.

1b) Wash Solution

Add the entire contents of the Wash Concentrate (100 mL) to approximately 2.0 L of CAP Class I water or the clinical laboratory reagent water, mix well, and adjust the final volume to 2.5 L.

1c) Diluent

Add the entire contents of the Diluent Concentrate (100 mL) to approximately 4.0 L of CAP Class I water or the clinical laboratory reagent water, mix well, and adjust the final volume to 5.0 L.

2) Calibration


2a) Calibration Curve

The calibrators for use with the ST AIA-PACK LH II have been standardized against WHO 2nd IS 80/552.

The calibration curve for the ST AIA-PACK LH II is stable for up to 90 days. Calibration stability is monitored by quality control performance and is dependent on proper reagent handling and AIA System maintenance according to the manufacturer's instructions.

Recalibration may be necessary more frequently if controls are out of the established range for this assay or if certain service procedures are performed (e.g. temperature adjustment, sampling mechanism changes, or detector lamp adjustment or change). For further information regarding instrument operation, consult the AIA System Operator's Manual.

The sample calibration curve below shows the algorithm used to calculate the results. This is an example only. Actual results will vary depending on the Instrument type and lot number used.

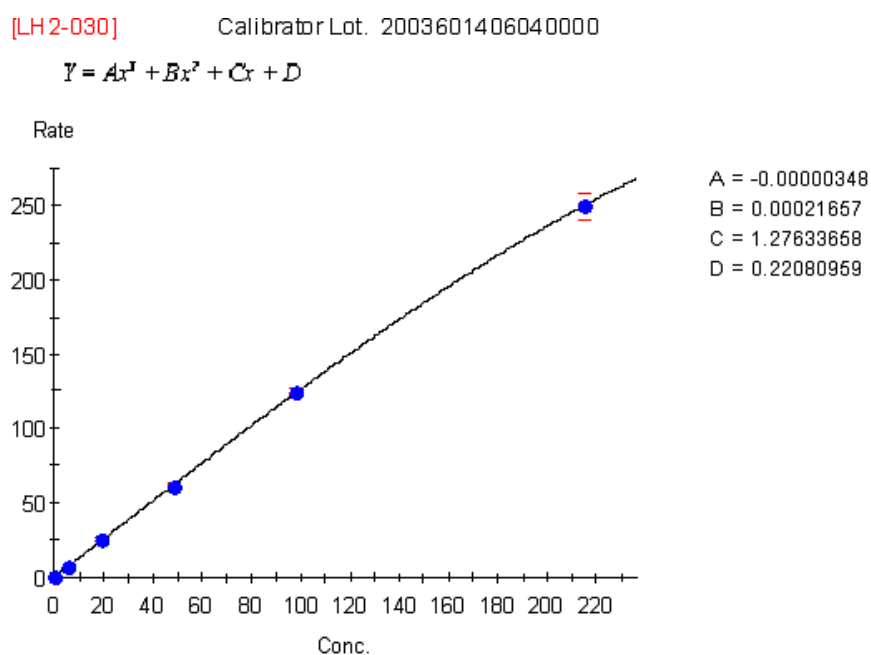

2b) Calibration Procedure

Refer to the appropriate AIA System Operator's Manual for procedural instructions.

- i) Verify that both the calibrator lot and concentration numbers have been correctly entered into the software.
- ii) The ST AIA-PACK LH II Calibrator (1) is provided ready for use.
- iii) The ST AIA-PACK LH II Calibrator (2) - (6) are lyophilized. Lyophilized calibrators should be reconstituted with 1.0 mL of CAP Class I water or the clinical laboratory reagent water.
- iv) Tosoh recommends that all calibrators be run in triplicate.

2c) Calibration Acceptability criteria

- i) The mean rate for the zero calibrator should be < 3.0 nM/sec.
- ii) Since there is a direct relationship between concentration and rate, the rates should increase as the concentration increases.
- iii) The replicate values should be within a 10% range.

2d) Calibration Review and Acceptance

- i) Review the calibration curve carefully, using the criteria listed above.
- ii) Edit the calibration if necessary, then accept the calibration.
- iii) For further information regarding calibration, consult the specific AIA System Operator's Manual.

3) Quality Control

3a) Commercially Available Controls

Commercially available controls should be run at least once per day. It is recommended that at least two (2) levels of controls, normal and abnormal, be used. Laboratory policy for this particular assay designates the following:

Control Material: _____
Frequency: _____

Lot number of control material, acceptable limits, and corrective action to be taken if controls do not meet laboratory criteria will be found in a separate quality control document maintained by the laboratory.

3b) Quality Control Procedure

- i) Assay quality control specimens as instructed in the specific Operator's Manual for the analyzer. In addition, refer to the AIA System Operator's Manual for detailed instructions on defining and editing the files.
- ii) Quality control material to be run with this assay is defined by individual laboratory policy.

4) Specimen Processing

4a) Preparation

Following specific instructions in the Operator's Manual for the analyzer, place samples on the instrument in the appropriate positions.

4b) Assay Procedure

- i) Verify a sufficient quantity of ST AIA-PACK LH II test cups for the number of samples to be run.
- ii) Load patient samples as instructed in the Operator's Manual and proceed with analysis. Note: The AIA-900 and AIA-2000 will require AIA-PACK Sample Treatment Cups if onboard dilutions are utilized.

Procedural Notes

1. Lyophilized Substrate must be completely dissolved.
2. Ligand assays performed by the Tosoh AIA System Analyzer require that the laboratory use water designated by the CAP Class I or by the clinical laboratory reagent water. Water should be tested at least once per month and should be free of particulate matter including bacteria. The pH of the water should also be routinely tested. For further information, consult the CLSI document GP40-A4-AMD, Preparation and Testing of Reagent Water in the Clinical Laboratory; Approved Guideline - Fourth Edition.
3. If a specimen Luteinizing Hormone concentration is found to be greater than the linearity limit of the assay, 200 mIU/mL; the specimen should be diluted with the AIA-PACK LH II Sample Diluting Solution and re-assayed according to the Assay Procedure. The recommended dilution for samples containing greater than the concentration of the highest calibrator is 1:10 or 1:100. However, it is desirable to dilute the sample so that the diluted sample reads between 10 and 150 mIU/mL. The dilution factor should be entered into the software. For further information on the dilution of specimens, refer to the AIA System Operator's Manual.
4. The AIA systems can store two different calibration curves for each analyte at one time. Therefore, up to two different lots of ST AIA-PACK LH II test cups can be used during the same run.
5. If the assay specifications for this test are not already in the system software, the specifications must be entered under test code 039.

Calculation of Results

The AIA Systems perform all sample and reagent handling operations automatically. The AIA Systems read the rate of fluorescence produced by the reaction and automatically convert the rate to luteinizing hormone concentration in mIU/mL.

For samples requiring dilution, the AIA-900 and AIA-2000 will automatically perform dilutions and calculate results if the dilution factors are entered into the software. For detailed information regarding programming dilutions, consult the appropriate Operator's Manual.

Evaluation of Results

Quality Control

In order to monitor and evaluate the precision of the analytical performance, it is recommended that commercially available control samples be assayed daily.

The minimum recommendations for the frequency of running internal control material are:

- After calibration, two levels of controls are run in order to accept the calibration curve.
- The two levels of controls are also repeated after calibration when certain service procedures are performed (e.g. temperature adjustment, sampling mechanism changes, maintenance of the wash probe or detector lamp adjustment or change).
- After daily maintenance, at least two levels of the control should be run in order to verify the overall performance of the Tosoh AIA System Analyzers.

If one or more control sample value(s) is out of the acceptable range, it will be necessary to investigate the validity of the calibration curve before reporting patient results.

Standard laboratory procedures should be followed in accordance with the regulatory agency under which the laboratory operates.

Limitations of the Procedure

For diagnostic purposes, the results obtained from this assay should be used in conjunction with other data (e.g., symptoms, results of other tests, clinical impressions, therapy, etc.).

Using ST AIA-PACK LH II, the highest concentration of Luteinizing Hormone measurable without dilution is approximately 200 mIU/mL, and the lowest measurable concentration is 0.2 mIU/mL (assay sensitivity).

Although the approximate value of the highest calibrator is 200 mIU/mL, the exact concentration may be slightly different. The assay specification, Assay Range High, should be defined as the upper limit of the assay range, 200 mIU/mL.

Although hemolysis has an insignificant effect on the assay, hemolyzed samples may indicate mistreatment of a specimen prior to assay and results should be interpreted with caution.

Lipemia has an insignificant effect on the assay except in the case of gross lipemia where spatial interference may occur.

Specimens from patients who have received preparations of mouse monoclonal antibodies for diagnosis or therapy may contain human anti-mouse antibodies (HAMA). Such specimens may show falsely elevated values when tested for Luteinizing Hormone.

Certain medications may interfere with assay performance. Specimens from patients taking medicines and/or medical treatment may show erroneous results. All results should be interpreted with respect to the clinical picture of the patient.

For a more complete understanding of the limitations of this procedure, please refer to the Specimen Collection and Handling, Warnings and Precautions, Storage and Stability, and Procedural Notes sections in this insert sheet.

Expected Values

Each laboratory should determine a reference interval which corresponds to the characteristics of the population being tested. As with all diagnostic procedures, clinical results must be interpreted with regard to concomitant medications administered to the patient.¹⁴

Reference Ranges

The ranges given here were determined in serum samples from 296 apparently healthy individuals.

Category	Samples	Range
Male:	132	1.7 – 11.2 mIU/mL
Ovulating Female:		
Follicular phase	52	1.7 – 13.3 mIU/mL
Peak	49	4.1 – 68.7 mIU/mL
Luteal phase	42	0.5 – 19.8 mIU/mL
Postmenopausal Female:	21	14.4 – 62.2 mIU/mL

Conversion Factors

LH concentrations in this application are in units of mIU/mL. Conversion to SI units of IU/L may be made using the following equation:

$$\text{IU LH/L} = \text{mIU LH/mL} \times 1.0$$

Performance Characteristics

1) Accuracy

1a) Recovery:

Three serum pools were spiked with three different levels of LH and assayed before and after spiking.

Sample	Initial Value (mIU/mL)	LH Added (mIU/mL)	Expected Value (mIU/mL)	Measured Value (mIU/mL)	Percent Recovery (%)
Serum A1	3.12	27.91	31.02	29.64	95.5
	3.12	55.81	58.93	60.39	102.5
	3.12	111.62	114.74	114.22	99.5
Serum B1	3.08	27.91	30.98	31.00	100.1
	3.08	55.81	58.89	61.33	104.1
	3.08	111.62	114.70	113.89	99.3
Serum C1	6.05	27.91	33.96	34.44	101.4
	6.05	55.81	61.86	64.19	103.8
	6.05	111.62	117.68	116.74	99.2

1a) Dilution:

Three serum samples containing high concentrations of LH were serially diluted with LH II Sample Diluting Solution and assayed.

Sample	Dilution Factor	Expected Value (mIU/mL)	Measured Value (mIU/mL)	Percent Recovery (%)
Serum A2	none		110.23	
	7.5/10	82.67	83.08	100.5
	5.0/10	55.11	56.41	102.3
	2.5/10	27.56	28.42	103.1
	1.0/10	11.02	10.50	95.3
Serum B2	none		92.93	
	7.5/10	69.69	75.12	107.8
	5.0/10	46.46	49.17	105.8
	2.5/10	23.23	24.61	105.9
	1.0/10	9.29	9.10	97.9
Serum C2	none		112.50	
	7.5/10	84.37	84.88	100.6
	5.0/10	56.25	58.15	103.4
	2.5/10	28.12	28.73	102.1
	1.0/10	11.25	10.86	96.5

2) Precision

2a) Intra-assay precision

The intra-assay (within run) precision was determined using three controls in a total of 20 runs. Within each run, one set of duplicates per control was assayed. The mean of each duplicate was used to obtain the pooled standard deviation (SD), which was then used to calculate the coefficient of variation (CV).

Sample	Mean (mIU/mL)	Pooled SD (mIU/mL)	Coefficient of Variation (%)
Control A3	4.94	0.123	2.5
Control B3	25.27	0.597	2.4
Control C3	98.14	1.733	1.8

2b) Inter-assay precision

The inter-assay (between run) precision coefficient of variation was evaluated at three different concentrations by analyzing control samples in 20 separate runs.

Sample	Number of Replicates	Mean (mIU/mL)	Standard Deviation (mIU/mL)	Coefficient of Variation (%)
Control A3	20	4.91	0.089	1.8
Control B3	20	25.17	0.461	1.8
Control C3	20	95.67	7.522	1.8

2c) Total precision

Total precision was determined by the duplicate assay of three controls in 20 separate runs. The means of each run were used to calculate the standard deviation (SD) and coefficient of variation (CV).

Sample	Mean (mIU/mL)	Pooled SD (mIU/mL)	Coefficient of Variation (%)
Control A3	4.94	0.134	2.7
Control B3	25.27	0.535	2.1
Control C3	98.14	2.032	2.1

Specificity

The following substances were tested for cross-reactivity. The cross-reactivity (%) is the percent of the compound which will be identified as LH. If these compounds are present in the specimen at the same concentration as LH, the final result will be increased by these percentages.

Compound	Cross-reactivity (%)
LH	100
TSH	0.37
FSH	0.0041
HCG	0.000072

Sensitivity

The minimal detectable concentration (MDC) of Luteinizing Hormone is estimated to be 0.2 mIU/mL. The MDC is defined as that concentration of LH which corresponds to the rate of fluorescence that is two standard deviations from the mean rate of fluorescence of 20 replicate determinations of a zero calibrator.

Interference

- Interference is defined, for purposes of this study, to be recovery outside of 10% the known specimen mean concentration.
- Hemoglobin (up to 430 mg/dL), free bilirubin (up to 17 mg/dL) and conjugated bilirubin (up to 19 mg/dL) do not interfere with the assay.
- Lipemia, as indicated by triglyceride (concentrations up to 1,660 mg/dL) does not interfere with the assay.
- Protein, as indicated by albumin (concentrations up to 50 g/dL) does not interfere with the assay.
- Ascorbic acid (concentrations up to 20 mg/dL) does not interfere with the assay.
- Tosoh Automated Immunoassays utilizing alkaline phosphatase-based technologies should not be used with samples from patients under Asfotase Alfa treatment.

References

1. Pierce, J.G., Eli Lilly Lecture: The Subunits of Pituitary Thyrotropin - Their Relationship to Other Glycoprotein Hormones. *Endocrinology* 1971, 89:1331.
2. Pierce, J.G., Bahl, O.P., Cornell, J.S., and Swaminathan, N., Biologically Active Hormones Prepared by Recombination of the Alpha Chain of Bovine Thyrotropin or of Bovine Luteinizing Hormone. *J. Biol. Chem.* 1971, 246:2321.
3. Amir, S.M., Dissociation of Glycoprotein Hormone: A Review of Recent Studies of Their Subunits. *Acta Endocrinologica*, 1972, 70:21.
4. Ward, D.N., McGregor, R.F., and Griffin, A.C., Chromatography of Luteinizing Hormone from Sheep Pituitary Glands. *Biochimica Et Biophysica Acta* 1959, 32:305.
5. Yen, S.S.C., Tsai, C.C., Naftolin, F., Vanderberg, G., and Ajabor, L., Pulsatile Patterns of Gonadotropin Release in Subject with and without Ovarian Function. *J. Clin. Endocrinol.* 1972. 34:671.
6. Naftolin, F., Yen, S.S.C., Perlman, D., Tsai, C.C., Parker, D.C., and Vargo, T., Nocturnal Patterns of Serum Gonadotropins During the Menstrual Cycle. *J. Clin. Endocrinol. Metab.* 1973, 37:6.
7. Collins, W.P., Branch, C.M., Collins, P.O., and Sallam, H.N., Biochemical Indices of the Fertile Period in Women. *Int. J. Fertil.* 1981, 26:196.
8. Pauerstein, C.J., Eddy, C.A., Croxatto, H.D., Hess, R., Siler-Khodr, T.M. and Croxatto, H.B., Temporal Relationships of Estrogen, Progesterone, and Luteinizing Hormone Levels to Ovulation in Women and Infrahuman Primates. *Am. J. Obstet. Gynecol.* 1978, 130:876.
9. Boyar, R., Perlow, M., Hellman, L., Kapen, S., and Weitzman, E., Twenty-four Hour Pattern of Luteinizing Hormone Secretion in Normal Men with Sleep Stage Recording. *J. Clin. Endocrinol. Metab.* 1972, 35:73.
10. Boyar, R.M., Katz, J., Finkelstein, J.W., Kapen, S., Weiner, H., Weitzman, E.D., and Hellman, L., Anorexia Nervosa: Immaturity of the 24-hour Luteinizing Hormone Secretory Pattern. *New Eng. J. Med.* 1974, 291:861.
11. Reyes, F.I., Koh, K.S., and Faiman, C., Fertility in Women with Gonadal Dysgenesis. *Am. J. Obstet. Gynecol.* 1976, 126:668.
12. Murdoch, A. P., Diggle, P.J., White, M. C., Kendall-Taylor, P., and Dunlop, W., LH in Polycystic Ovary Syndrome: Reproducibility and Pulsatile Secretion. *J. Endocrinol.* 1989, 121:185.
13. Obhrai, M., Lynch, S. S., Holder, G., Jackson, R., Tang, L., and Butt, W. R., Hormonal Studies on Women with Polycystic Ovaries Diagnosed by ultrasound. *Clin. Endocrin.* 1990, 32:467.
14. Haesslein, H.C., and Lamb, E.H., Pituitary Tumors in Patients with Secondary Amenorrhea. *Am. J. Obstet. Gynecol.* 1976, 125:759.
15. DeVane, G.W., Czekala, N.M., Judd, H.L., and Yen, S.S.C., Circulating Gonadotropins, Estrogens, and Androgens in Polycystic Ovarian Disease. *Am. J. Obstet. Gynecol.* 1975, 121:496.

**TOSOH CORPORATION****Bioscience Division**

2-2-1 Yaesu, Chuo-ku, Tokyo 104-0028 Japan

Phone: +81 3 6636 3734

Fax: +81 3 6636 3627

Website: www.tosoh.com**Supplied by****TOSOH BIOSCIENCE, INC.**3600 Gantz Road,
Grove City, OH 43123 USA

Phone: +1 650 615 4970

Fax: +1 650 615 0415

Phone: (800) 248 6764

Fax: (800) 685 7595

Website: www.tosohbioscience.us

European Conformity



In vitro diagnostic medical device



Consult instructions for use



Temperature limitation



Batch code / Lot number



Manufacturer

Authorized representative
in the European Community


Use by date

Catalogue number
/ Part number

Supplied by



Sufficient for

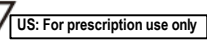

US: For prescription use only

AIA-PACK

ST LH II Calibrator Set

Intended Use

The ST AIA-PACK LH II Calibrator Set is intended for IN VITRO diagnostic use only for the calibration of the ST AIA-PACK LH II Assay.

Summary and Explanation

The ST AIA-PACK LH II Calibrator Set contains human serum with assigned levels of Luteinizing Hormone. Calibration should be performed according to the schedule indicated in the AIA System Operator's Manual. The calibrators in this set have been standardized against WHO 2nd IS 80/552.

Material Provided (Cat. No. 0025396)

2 x 1 mL	Calibrator No. 1	0	mIU/mL
----------	------------------	---	--------

Human serum containing no detectable concentration of LH (0 mIU LH/mL), and 0.1% sodium azide as preservative. (Ready for Use)

2 x 1 mL	Calibrator No. 2	5	mIU/mL (approx.)
	No. 3	20	mIU/mL (approx.)
	No. 4	50	mIU/mL (approx.)
	No. 5	100	mIU/mL (approx.)
	No. 6	200	mIU/mL (approx.)

Human serum containing the assigned concentration of LH (described on each vial), and 0.1% sodium azide as a preservative. (Lyophilized)

Warnings and Precautions

- The ST AIA-PACK LH II Calibrator Set is for in vitro diagnostic use.
- These materials contain sodium azide, which may react with lead or copper plumbing to form potentially explosive metal azides. When disposing of such reagents, always flush with large volumes of water to prevent azide build-up.
- Human sera used in the preparation of this product has been tested by FDA cleared methods and found negative for the presence of HBsAg and antibody to HIV-1 and HCV. Because no test method can offer complete assurance that products derived from human blood will not transmit infectious agents, it is recommended that this product be handled with the same precautions as used for patient samples.
- Do not use beyond the expiration date.

Preparation and Storage

- Using a volumetric pipette or a calibrated adjustable pipette, reconstitute the lyophilized calibrators accurately with 1.0 mL of CAP Class I water or the clinical laboratory reagent water. Allow the lyophilized material to fully dissolve, then mix the calibrators gently but thoroughly prior to performing the calibration.
- Bring calibrator to room temperature (18 - 25 °C) for use.
- Always store the Calibrator Set in an upright position at 2 - 8 °C when not in use.

Stability

When stored unopened and refrigerated at 2 - 8 °C, the ST AIA-PACK LH II Calibrator Set is stable until the expiration date on the label. After opening, the calibrators should be used within 24 hours.

Procedure

Refer to the CALIBRATION PROCEDURE in the AIA-PACK section of this analyte application. For additional procedural instructions regarding calibration, refer to the AIA System Operator's Manual.

1. Verify that both the calibrator lot and concentrations have been correctly entered into the analyzer software.
2. Add the appropriate amounts of each calibrator to sample cups (refer to the instrument worklist for the amount required in each sample cup.)
3. Place the sample cups and the test cups on the analyzer as indicated on the worklist.
4. Tosoh recommends that all calibrators be run in triplicate.

Assignment of Values

The ST AIA-PACK LH II Calibrator Set contains assigned concentrations of Luteinizing Hormone. The assigned value is determined on a lot-to-lot basis and is designed to provide an assay calibration range of approximately 0 to 200 mIU/mL of Luteinizing Hormone. The calibrators in this set have been standardized against WHO 2nd IS 80/552.

Results

- 1) The mean rate for the zero calibrator should be < 3.0 nM/sec.
- 2) Since there is a direct relationship between concentration and rate, the rates should increase as the concentration increases.
- 3) The replicate values should be within a 10% range.

Limitations


The ST AIA-PACK LH II Calibrator Set is designed solely for use with AIA-PACK assay procedures.

USE ONLY THIS CALIBRATOR SET WITH ST AIA-PACK LH II (CAT. NO. 0025296).

References

1. AIA Analyte Application Manual (AAM). Tosoh Bioscience, Inc., Grove City, OH.
2. AIA-System Operator's Manual. Tosoh Bioscience, Inc., Grove City, OH.

	TOSOH CORPORATION Bioscience Division 2-2-1 Yaesu, Chuo-ku, Tokyo 104-0028 Japan Phone: +81 3 6636 3734 Fax: +81 3 6636 3627 Website: www.tosoh.com
Supplied by	TOSOH BIOSCIENCE, INC. 3600 Gantz Road, Grove City, OH 43123 USA Phone: +1 650 615 4970 Fax: +1 650 615 0415 Phone: (800) 248 6764 Fax: (800) 685 7595 Website: www.tosohbioscience.us

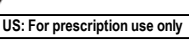 European Conformity	 In vitro diagnostic medical device	 Consult instructions for use	 Temperature limitation
 Batch code / Lot number	 Manufacturer	 Authorized representative in the European Community	 Use by date
 Catalogue number / Part number	 Supplied by	 Sufficient for	 US: For prescription use only

AIA-PACK

LH II Sample Diluting Solution

Intended Use

The AIA-PACK LH II Sample Diluting Solution is intended for IN VITRO DIAGNOSTIC USE ONLY to dilute patient samples that have concentrations of Luteinizing Hormone above the linear range of the assay.

Summary and Explanation

The AIA-PACK LH II Sample Diluting Solution contains human serum with no detectable concentration of Luteinizing Hormone. This Sample Diluting Solution is to be used only with samples that are being tested for Luteinizing Hormone concentrations using the ST AIA-PACK LH II assay.

Material Provided (Cat. No. 0020596)

4 x 4 mL Sample Diluting Solution

Human serum containing no detectable concentration of Luteinizing Hormone (0 mIU LH/mL), and 0.1% sodium azide as a preservative.

Warnings and Precautions

- The AIA-PACK LH II Sample Diluting Solution is for in vitro diagnostic use.
- These materials contain sodium azide, which may react with lead or copper plumbing to form potentially explosive metal azides. When disposing of such reagents, always flush with large volumes of water to prevent azide build-up.
- Human sera used in the preparation of this product has been tested by FDA cleared methods and found negative for the presence of HBsAg and antibody to HIV-1 and HCV. Because no test method can offer complete assurance that products derived from human blood will not transmit infectious agents, it is recommended that this product be handled with the same precautions as used for patient samples.
- Do not use beyond the expiration date.

Preparation and Storage

- The AIA-PACK LH II Sample Diluting Solution is provided ready to use.
- Always store the Sample Diluting Solution in an upright position at 2 - 8 °C when not in use.

Stability

When stored unopened and refrigerated at 2 - 8 °C, the AIA-PACK LH II Sample Diluting Solution is stable until the expiration date on the label. After opening, the Sample Diluting Solution is stable for up to 7 days when refrigerated at 2 - 8 °C.

Procedure

Refer to the AIA System Operator's Manual for additional procedural instructions regarding sample dilution.

1. If a specimen is found to contain greater than the linearity limit of approximately 200 mIU/mL, the specimen should be diluted with the Sample Diluting Solution and assayed according to the Procedure in the AIA-PACK section of the instructions for use.
2. The AIA-900 and AIA-2000 will perform dilutions automatically if the dilution factors are entered into the software prior to assaying the diluted sample.
3. The recommended dilution for specimen containing greater than the concentration of the highest calibrator is 1:10 or 1:100. However, it is desirable to dilute the samples so that the diluted sample reads between 15 and 150 mIU/mL.

Results

When an auto-dilution is performed, the AIA instrument will calculate the final result.

Limitations

The AIA-PACK LH II Sample Diluting Solution is designed solely for use with AIA-PACK assay procedures.

References

1. AIA Analyte Application Manual (AAM). Tosoh Bioscience, Inc., Grove City, OH.
2. AIA System Operator's Manual Tosoh Bioscience, Inc., Grove City, OH.

**TOSOH CORPORATION****Bioscience Division**

2-2-1 Yaesu, Chuo-ku, Tokyo 104-0028 Japan

Phone: +81 3 6636 3734

Fax: +81 3 6636 3627

Website: www.tosoh.com**Supplied by****TOSOH BIOSCIENCE, INC.**3600 Gantz Road,
Grove City, OH 43123 USA

Phone: +1 650 615 4970

Fax: +1 650 615 0415

Phone: (800) 248 6764

Fax: (800) 685 7595

Website: www.tosohbioscience.us

European Conformity



In vitro diagnostic medical device



Consult instructions for use



Temperature limitation



Batch code / Lot number



Manufacturer

Authorized representative
in the European Community

Use by date

Catalogue number
/ Part number

Supplied by



Sufficient for

US: For prescription use only